

Proloquo2Go

Making it work for you!



Part 3

Specific customisations in edit mode

Customising in Options

Backing up & Restoring

Where to get more information



Specific customisations
in edit mode



Home 1 2

I	= is	what?	where?	not	more
you	want	do	have	on	all done

Arrange Properties Folder Done

Background Color Default >

Border Color Default >

Border Width Default >

Text Options Default >

Display Image and Label >

Visibility Normal >

Voice Properties

Voice Default >

Speech Rate Default >

Voice Personalization Default >

Edit properties of a group of buttons






I

=

is

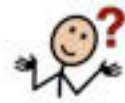


what



where



not

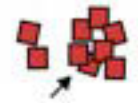

more



you



want



do



have



on



all done

Background Color

Pastel Colors



Pastel Red



Pastel Violet




Pastel Pink



Pastel Purple



Pastel Blue

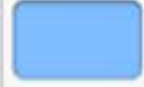

Pastel Aqua




Pastel Cyan




Pastel Sea Green



Pastel Green

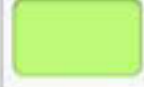

Pastel Lime Green

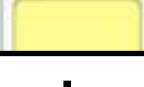

Pastel Yellow



Edit background colour of a group of buttons

Home

12

I

is

what

where

not

more

you

want

do

have

on

all done

Properties

Display

Use Default Choice


Image and Label

Image

Label

Image and Label

Edit the display properties for a group of buttons

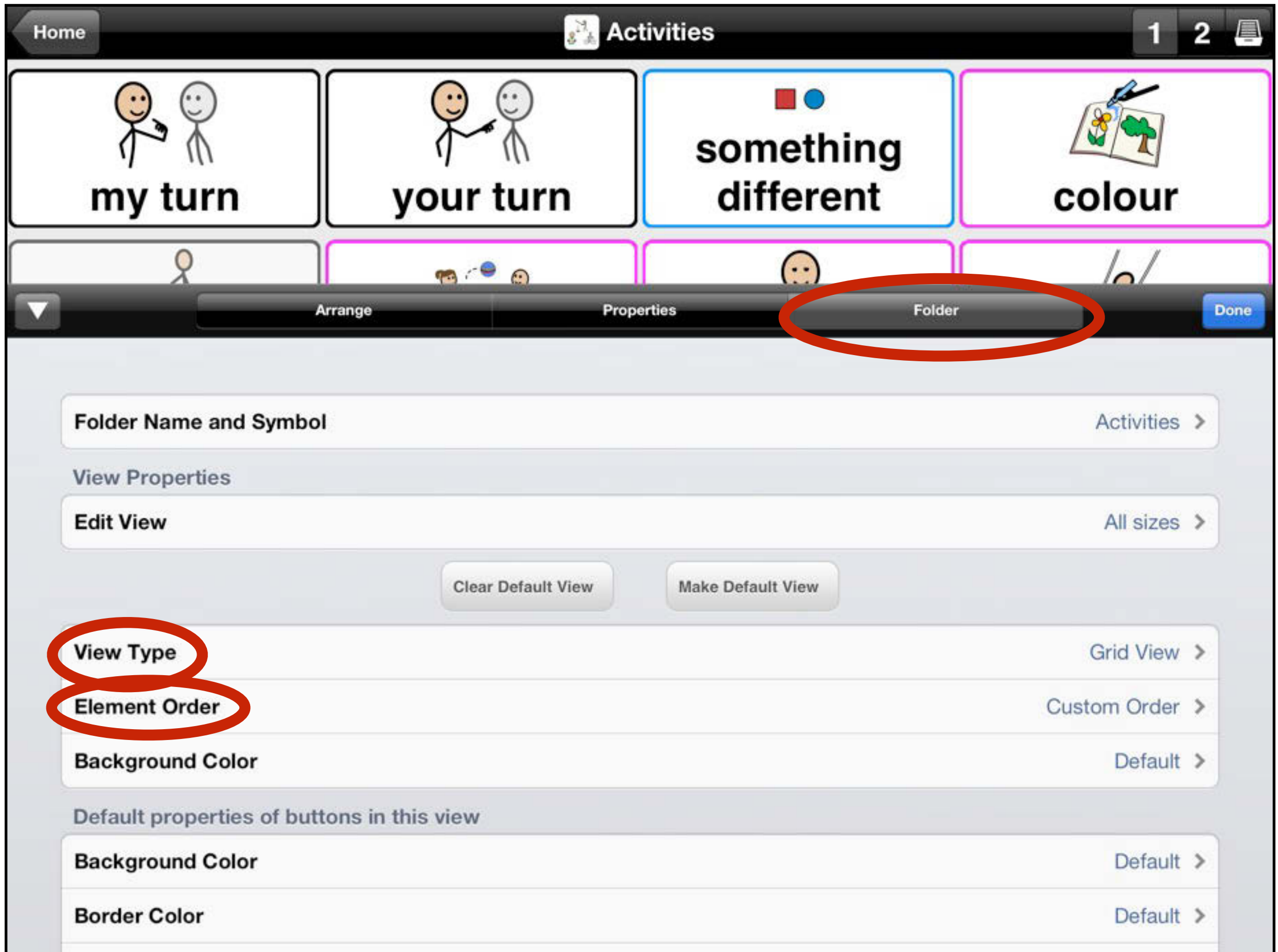


Edit folder options




Social stories

Calming down


12

 when I get excited


 I need to calm down

123

I count to ten

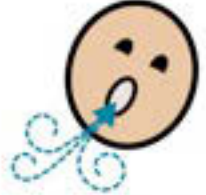 I take 3 deep breaths

SIGH

 Sigh! Sigh! Sigh!

▲

Arrange

Properties

Folder

Done

View Type

Element Order

Background Color

List View >

Custom Order >

Default >


Edit folder options - grid/list view





Home


Activities

12

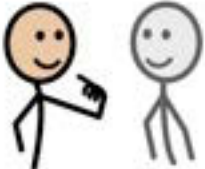
my turn

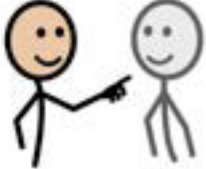
your turn


something different


colour

▼

Arrange

Properties

Folder

Done

Folder Name and Symbol

Activities >

View Properties

Edit View

All sizes >

Clear Default View

Make Default View

View Type

Grid View >

Element Order

Custom Order >

Background Color

Default >

Default properties of buttons in this view

Background Color


Default >

Border Color

Default >

Edit view for this page only



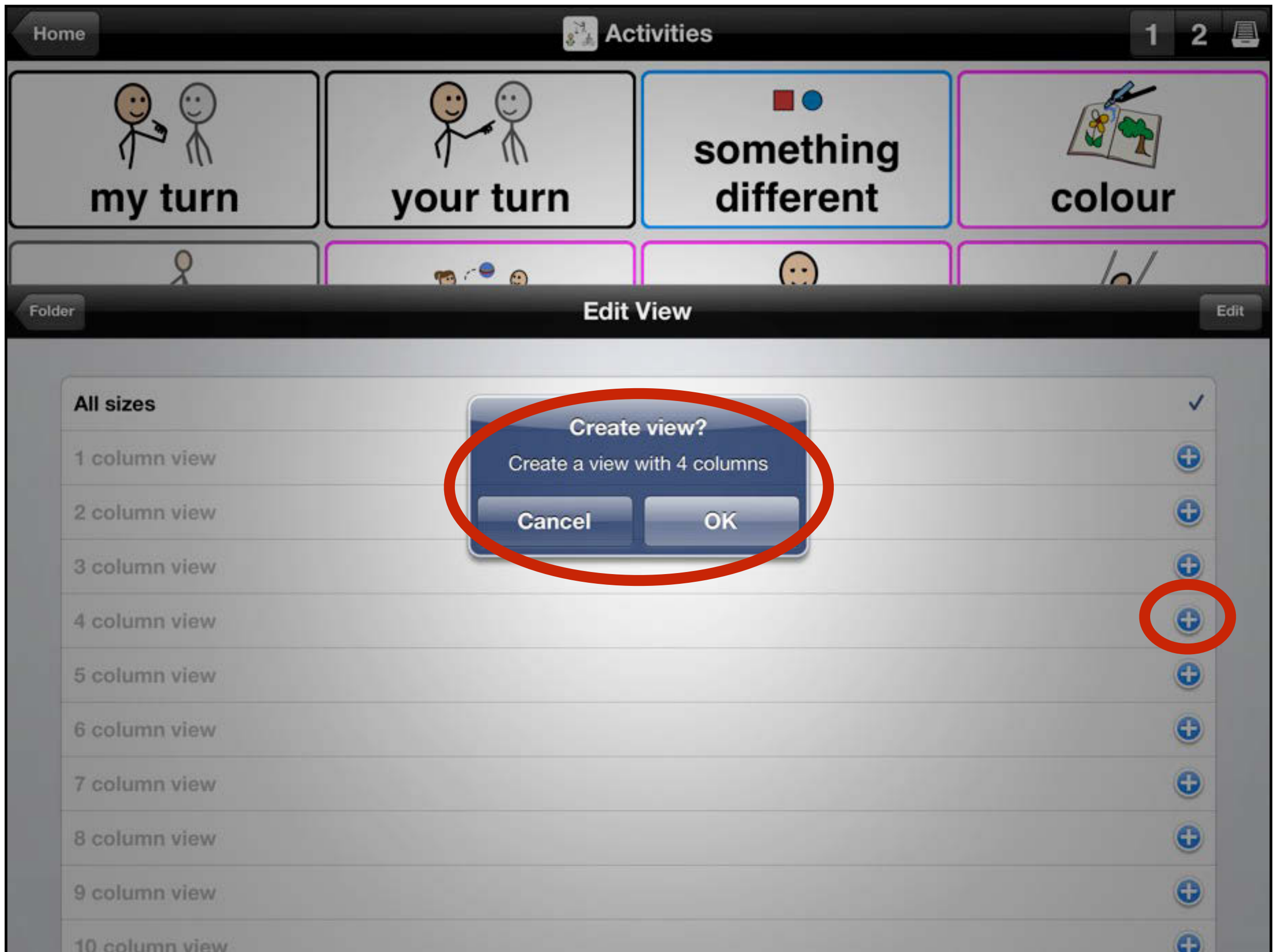

Create view (number of columns) for this page only



Home Activities 1 2

my turn your turn something different colour

Arrange Properties Folder Done

Folder Name and Symbol Activities >

View Properties

Edit View 4 column view >

OK

Clear Default View Make Default View

View Type Grid View >

Element Order Custom Order >

Background Color Default >

Default properties of buttons in this view

Background Color Default >

Border Color Default >


Make this column view the default for this page only




Home

Activities


12

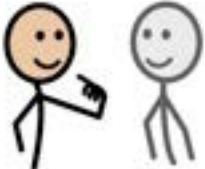


my turn

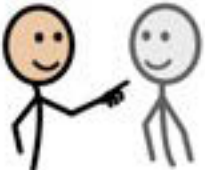


your turn




something different




play catch




read




listen to music




go for a walk




colour



swing



play outside



tickle

▲

Arrange

Properties

Folder

Done

View Type

Grid View >

Element Order

Free Positioning >

Background Color

Default >

Free positioning- move the buttons to anywhere on the page


Customising in Options






Options Menu



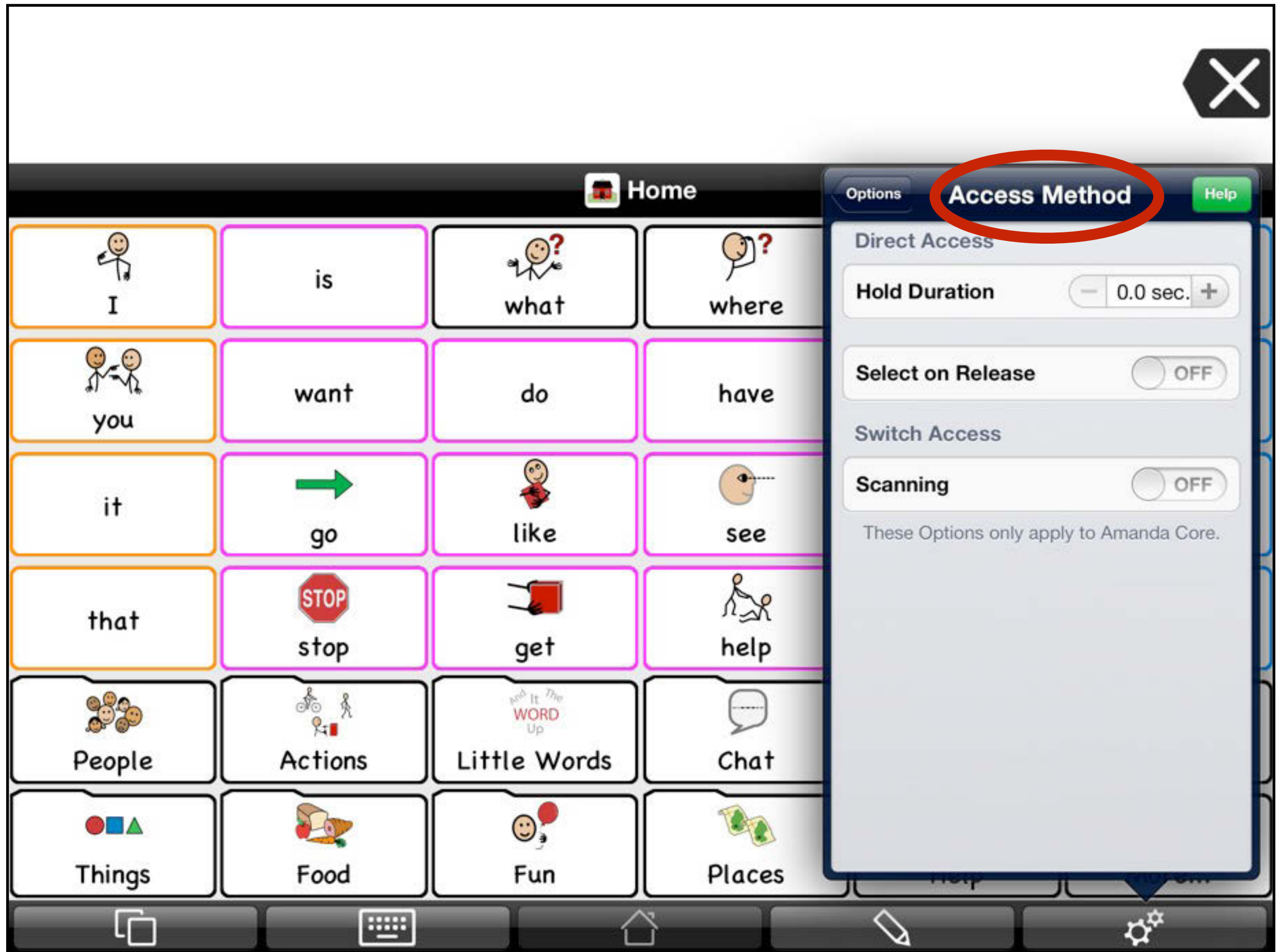


Options - Access Method



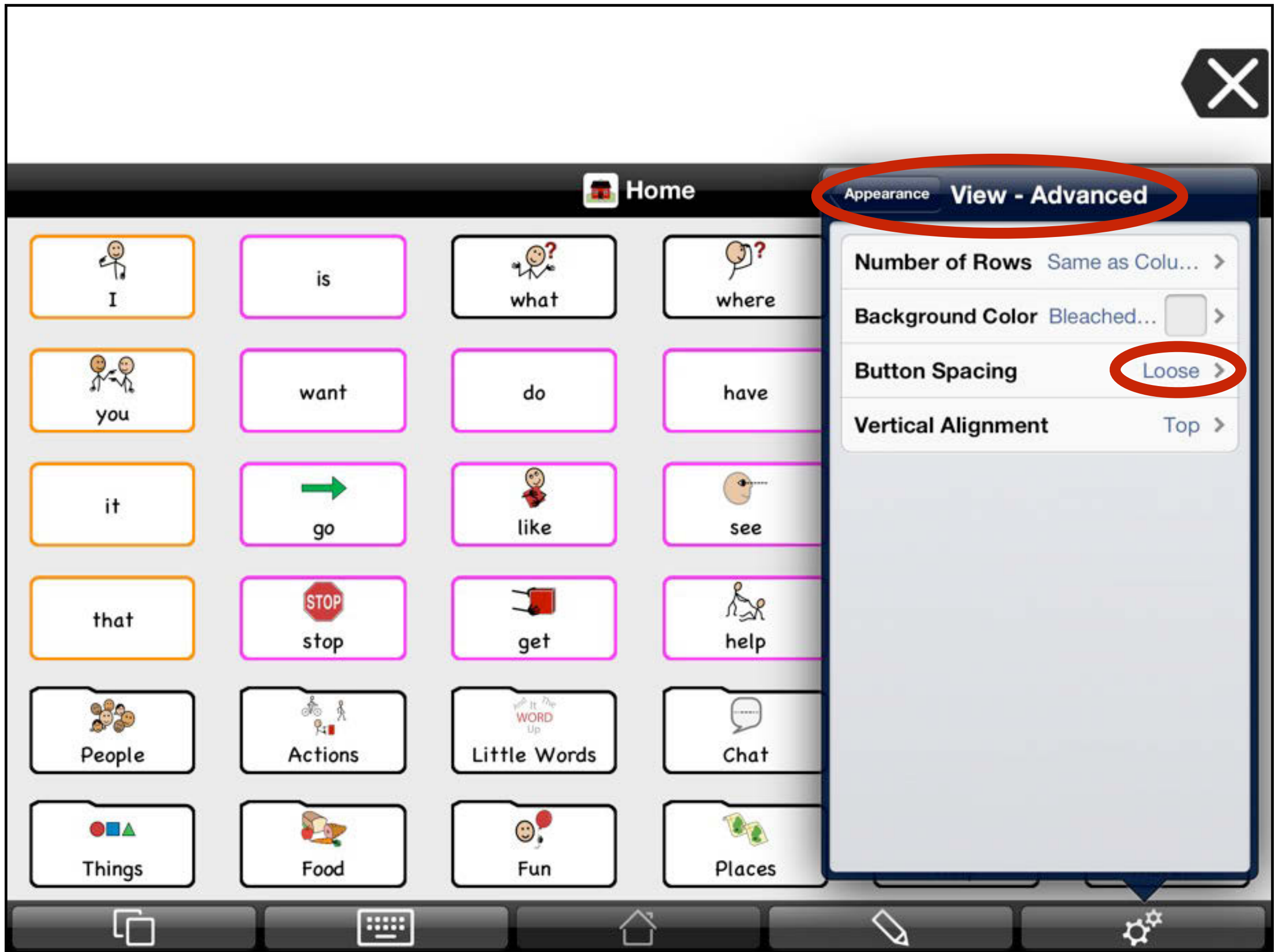


Options - Appearance (may help access)

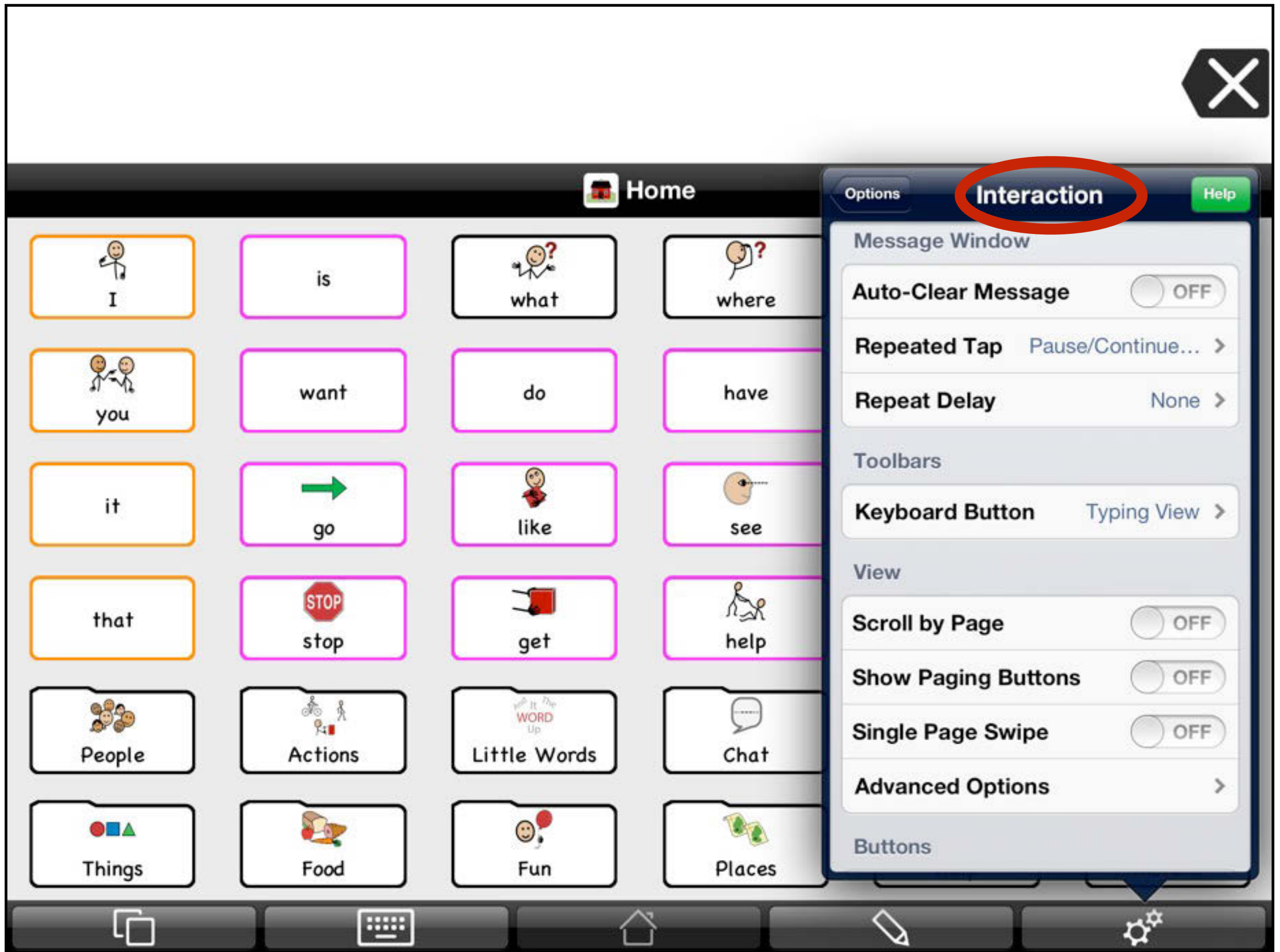


Options - Interaction



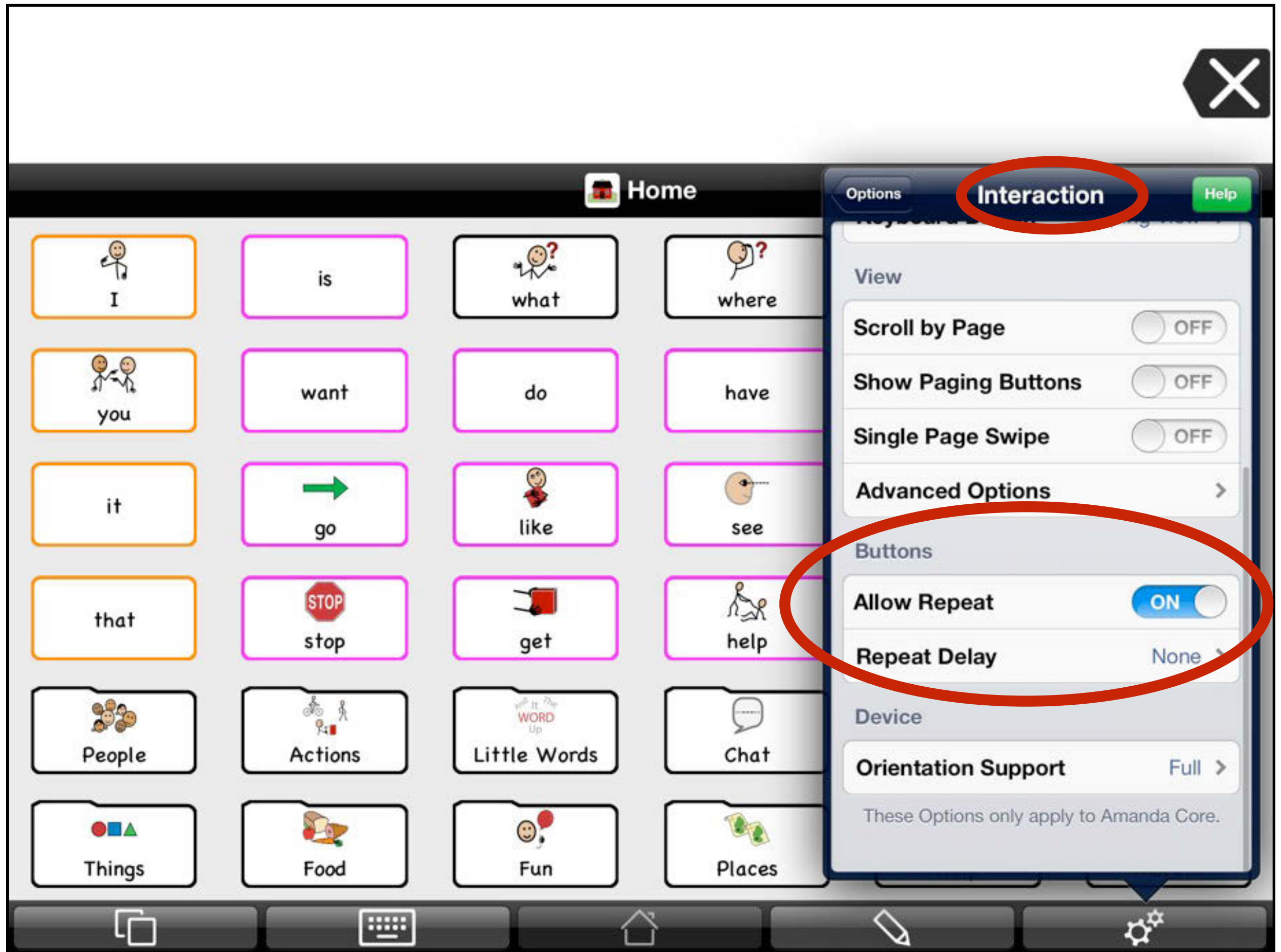


Options - Interaction - buttons repeat



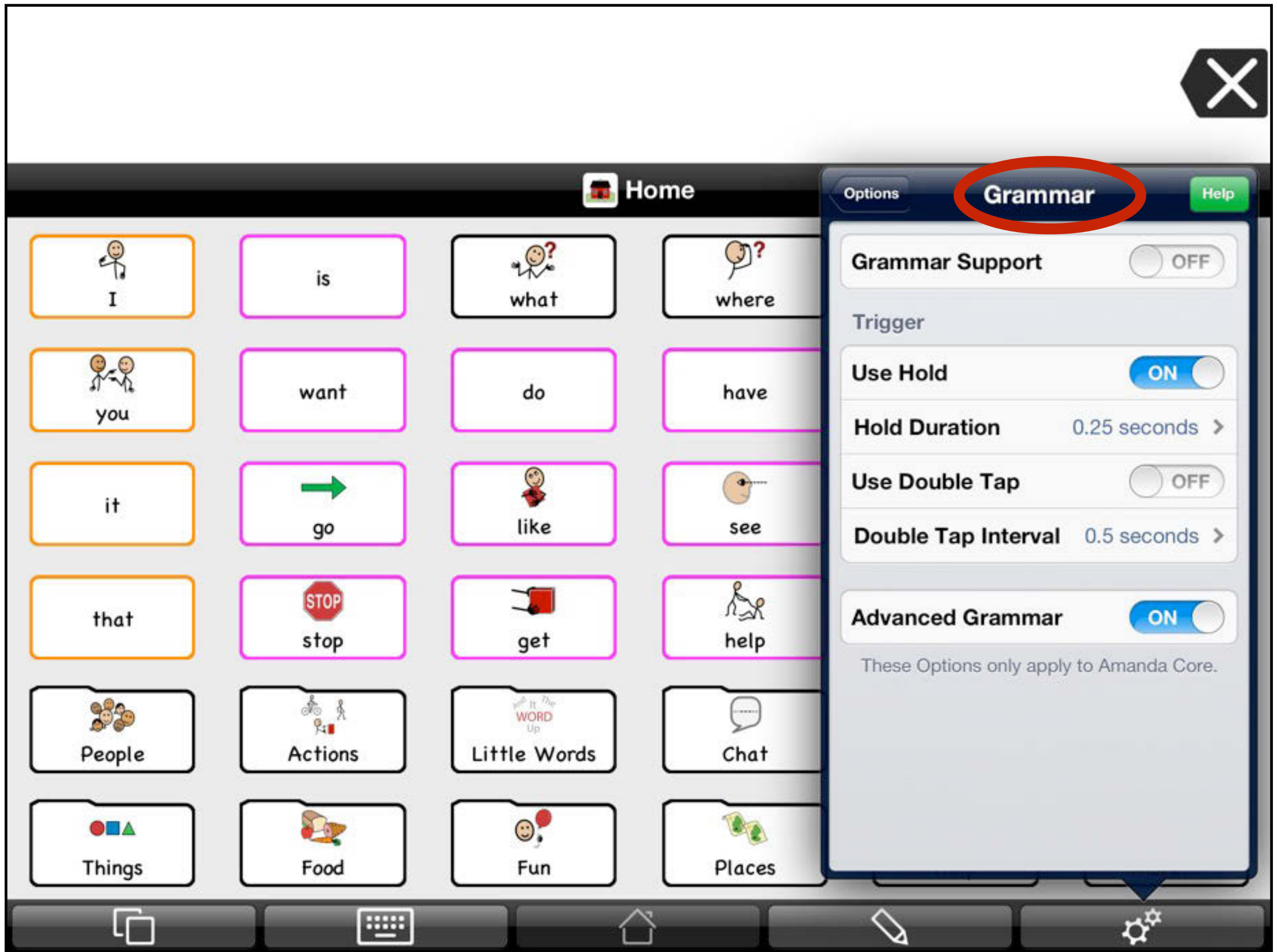


Options - Grammar



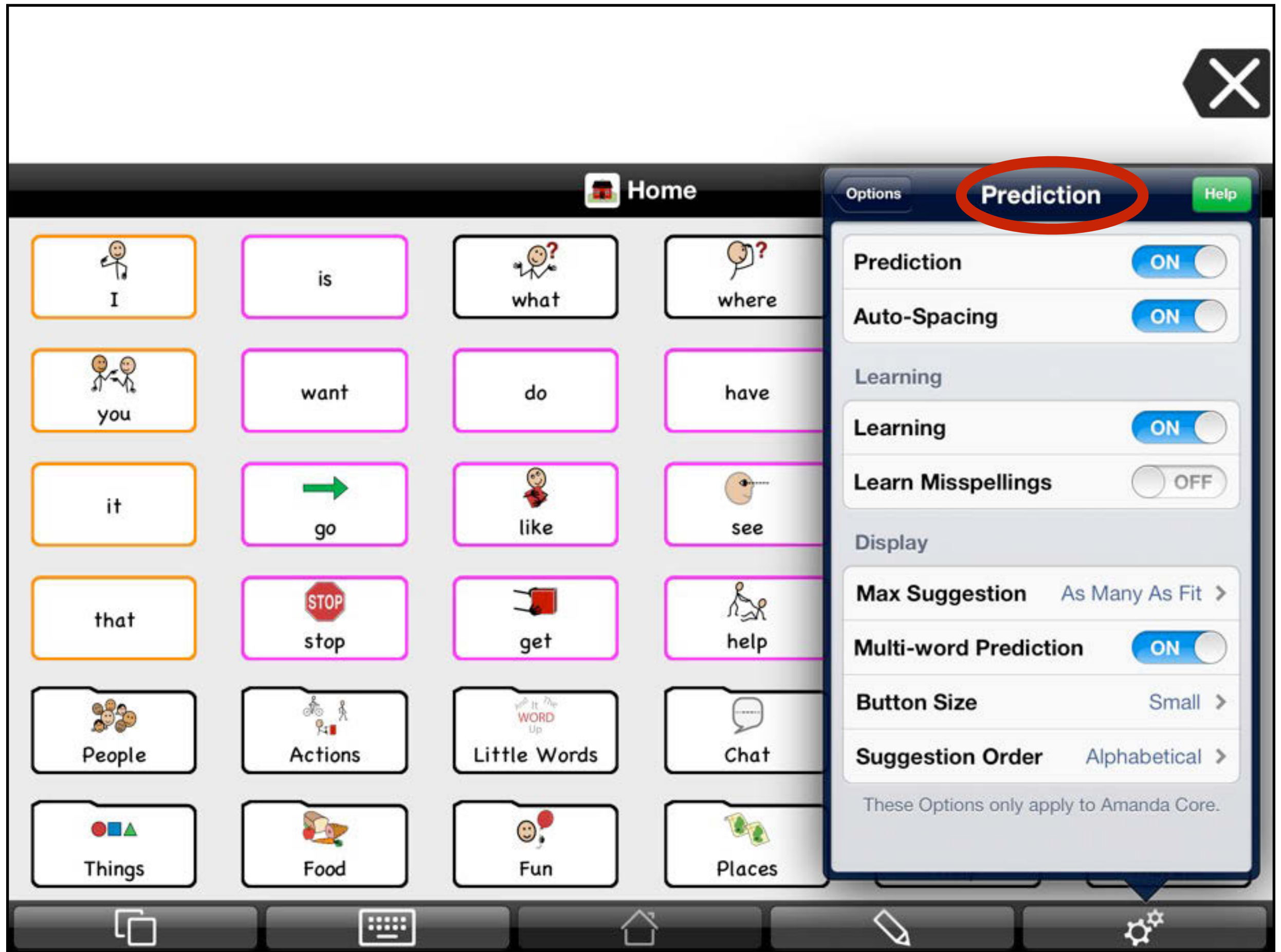


Options - Prediction



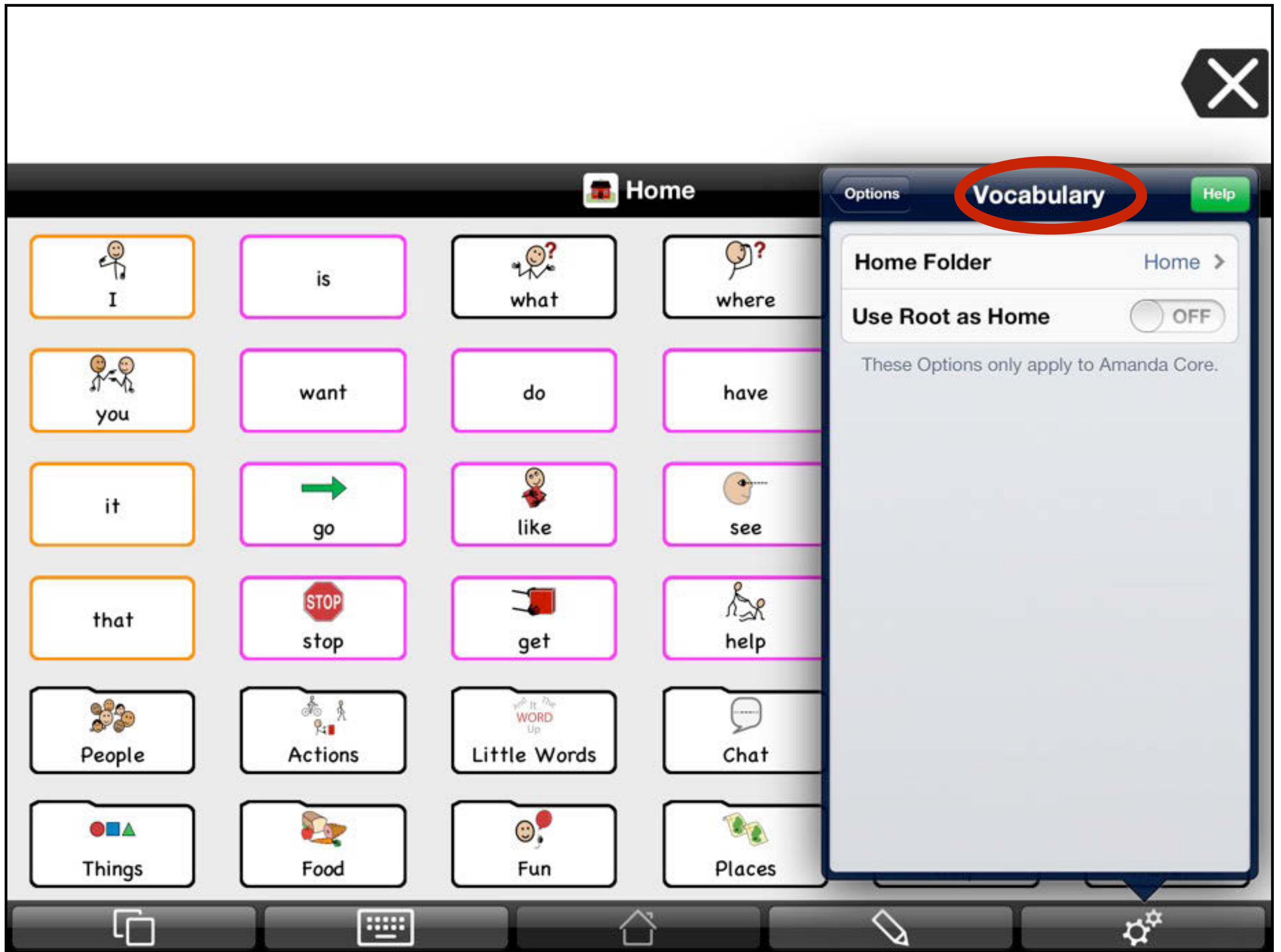


Options - Vocabulary



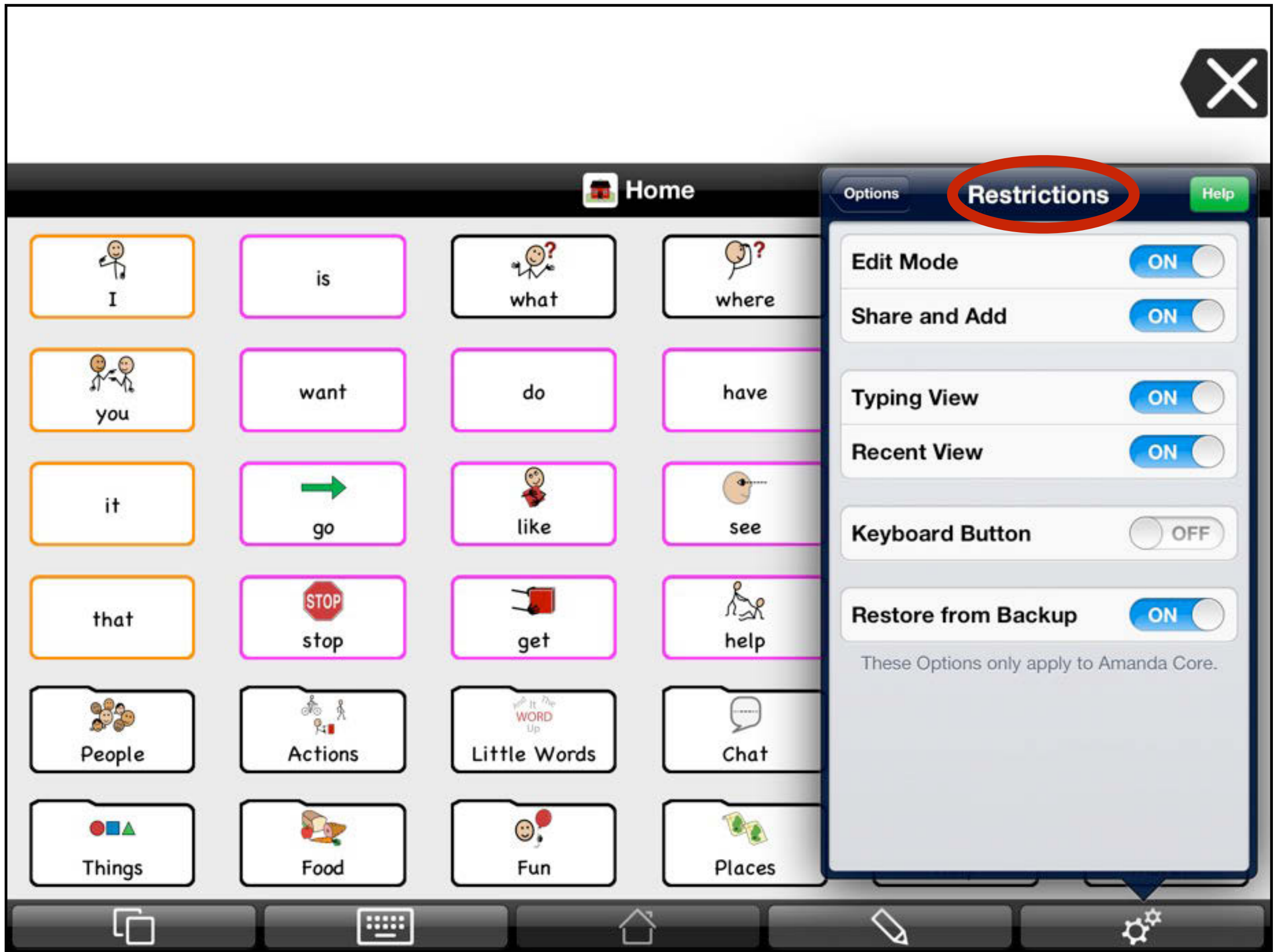


Options - Restrictions



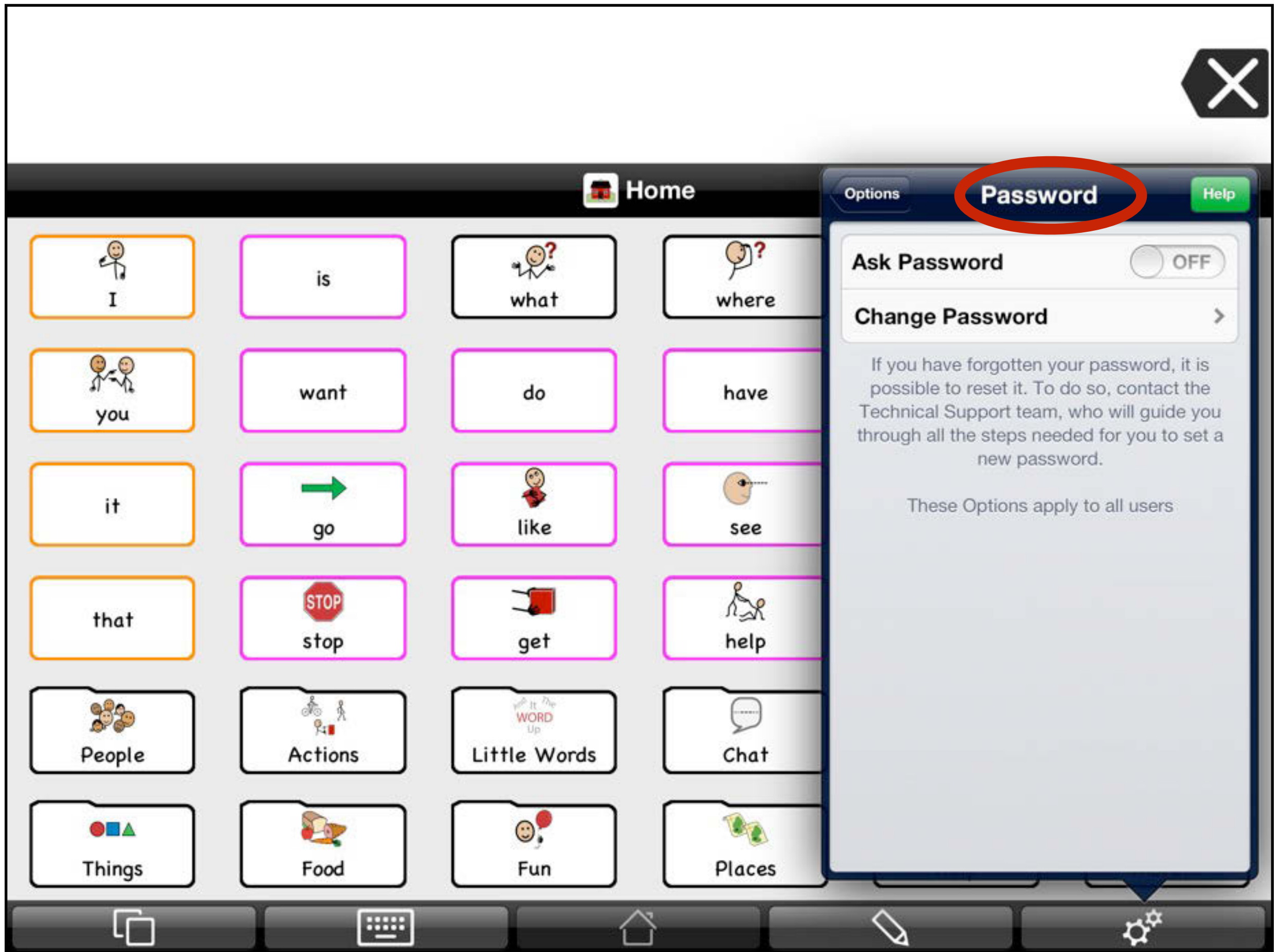

Options - Password



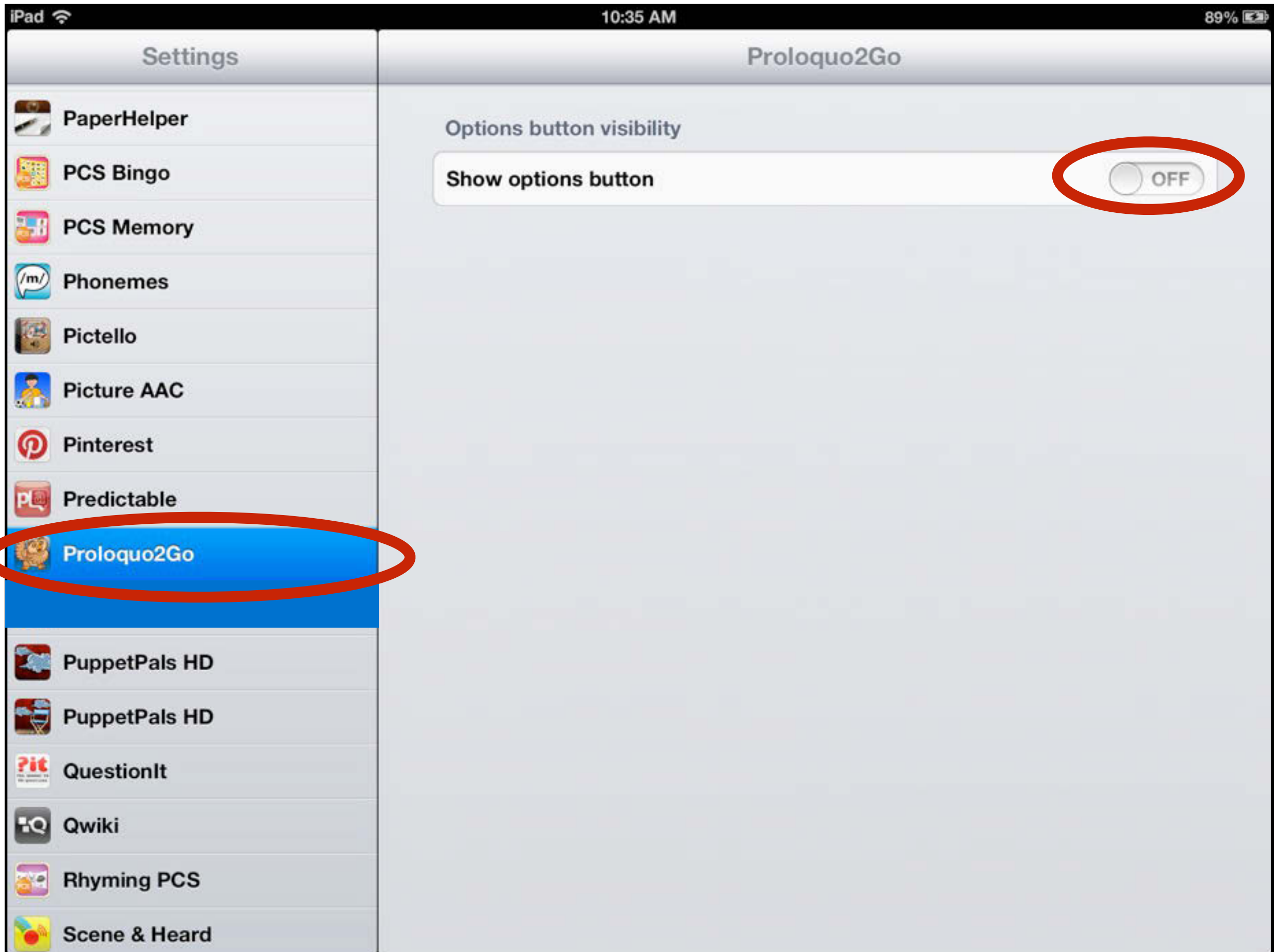

Outside the app- Settings- turn Options off



Backing up & Restoring





Options - Backup



3 ways to backup

- DropBox
- Wifi- Connect to Computer
- File Sharing on iTunes



Back up website links

- <http://www.assistiveware.com/product/proloquo2go/downloads>
- <http://www.spectronicsinoz.com/blog/apps-and-mobile-learning/proloquo2go-backing-up-and-restoring/>



**Where to get
more information**



 facebook.com/Spectronics

 twitter.com/Spectronics

 Spectronics.com.au/Blog

Spectronics P2Go Facebook page



Proloquo2Go Professionals



Proloquo2Go Parents



Access our website for resources

<http://bit.ly/spectronicsp2go>




Proloquo2Go

Proloquo2Go® is an award-winning Augmentative and Alternative Communication (AAC) solution for iPad, iPhone and iPod touch for people who have difficulty speaking or cannot speak at all. Providing a “voice” to over 50,000 individuals around the world, Proloquo2Go enables people to talk using symbols or typed text in a natural-sounding voice that suits their age and character.



User Videos

Watch the five Proloquo2Go user videos:



Enrique is a 9-years-old boy who has Down Syndrome and does not speak. He now uses Proloquo2Go to communicate.



SpectronicsOnline

Subscription

spectronics.com.au/online



INCLUSIVE LEARNING TECHNOLOGIES CONFERENCE 2014

20-23 May 2014

Gold Coast | Queensland | Australia

spectronics.com.au/conference

Join the Spectronics team for four full-on days of immersion in the world
of technologies to support diverse learners!



Six awesome **Keynote Presenters** now confirmed!



Kevin Honeycutt

Educational Technology Integration Evangelist

Carole Zangari

AAC in Special Ed and Autism Specialist

Dave Edyburn

Special Education Technology Research Publisher

Jason Carroll

Instructional Technology Consultant and UDL Advocate

Jason Gibson

Learning Support Specialist focused on Research-based Strategies

Greg O'Connor

Australian Educator Specialising in Support of Diverse Learners

Upcoming Workshops

Brisbane | Sydney | Melbourne
August - September 2013

iPad MasterClass

Supporting younger
students with autism

iPad MasterClass

Supporting older students
with literacy difficulties





Calendar



Maps



Mail



Apple Remote



Pages



Flickr



Camera



Notes



Facetime



Evernote



Youtube



Twitter



iBooks



App Store

The iPad Sandpit!



Safari



iPod



Photos



Settings



Amanda Hartmann
Inclusive Technology Consultant
(Speech Pathologist)



amandah@spectronics.com.au